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## Infusion preparation (Intrafix® SafeSet)

It's in your hands – Infection prevention in  
veterinary medicine

# Infusion preparation

Disinfect work surface (e.g., Meliseptol<sup>®</sup> acute) and hands (Softa-Man<sup>®</sup> pure).

Prepare work equipment. Label infusion. Remove protective film. Perform spray or wipe disinfection of the rubber plug if necessary.

Use the spike of the Intrafix<sup>®</sup> SafeSet to pierce the rubber stopper (without rotating).

Close roller clamp and fill drip chamber. Electrolyte solutions 1/3-1/2; transfusions > 2/3. When using Intrafix<sup>®</sup> SafeSet, wait a few seconds to allow the AirStop filter to unfold.

Open the roller clamp and fill the line.

When using Intrafix<sup>®</sup> SafeSet, the PrimeStop membrane eliminates the need to remove the cap, so the Luer-Lock connector remains sterile.

Close roller clamp.

# Infusion preparation

Perform hygienic hand disinfection.

Wipe- or spray-disinfect the CareSite® valve.

Connect infusion.

Additional information, this poster, and much more can be found at:  
[www.bbraun-vetcare.com/inyourhands](http://www.bbraun-vetcare.com/inyourhands)

B. Braun Vet Care GmbH | Am Aesculap-Platz | 78532 Tuttlingen | [info@bbraun-vetcare.de](mailto:info@bbraun-vetcare.de) | [www.bbraun-vetcare.com](http://www.bbraun-vetcare.com)

Thank you very much – The pictures were taken in Hergolding with the generous support of the small animal specialists Frontier GmbH.

The product trademarks "CareSite", "Intrafix", "Meliseptol" and "Softa-Man" are registered trademarks of B. Braun Melsungen AG. Subject to technical alterations. This brochure may only be used as information about our products. Reproduction in whole or in part is not permitted.