



Medication administration (Injekt® / MiniSpike® 2)

It's in your hands – Infection prevention in
veterinary medicine



Medication administration



Wipe-disinfect work surfaces.
Perform hygienic hand disinfection (Softa-Man® pure). Wear gloves if necessary (depending on the medication).



Prepare work equipment. Wipe- or spray-disinfect the rubber stoppers or MiniSpike®s.




When using a MiniSpike® 2, attach the syringe to the valve securely, then invert the medication and syringe (Inject®) and withdraw the medication.



Close the Injekt® syringe with a sterile combination stopper.



Transport the medication to the patient in a disinfected kidney dish.

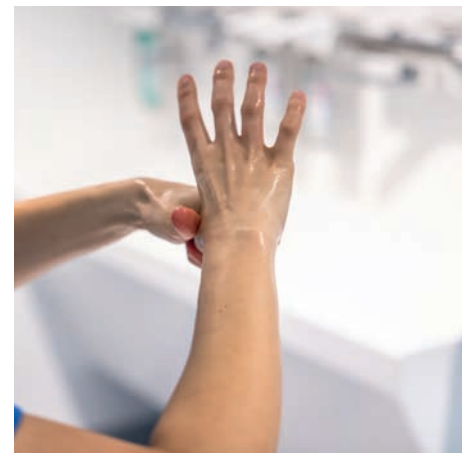

Medication administration is an aseptic activity and requires hygienic hand disinfection.

Medication administration



Wipe- or spray-disinfect the CareSite® valve before administering medication.



Medication administration and rinsing with sterile 0.9% NaCl solution.



Additional information, this poster, and much more can be found at:
www.bbraun-vetcare.com/inyourhands

B. Braun Vet Care GmbH | Am Aesculap-Platz | 78532 Tuttlingen | info@bbraun-vetcare.de | www.bbraun-vetcare.com

Thank you very much – The pictures were taken in Hergolding with the generous support of the small animal specialists Frontier GmbH.

The product trademarks "CareSite", "Injekt", "MiniSpike" and "Softa-Man" are registered trademarks of B. Braun Melsungen AG. Subject to technical alterations. This brochure may only be used as information about our products. Reproduction in whole or in part is not permitted.