Prontoderm® C

Read instruction for use.
Vor Anwendung Gebrauchsinformation lesen.
Consultez les instructions d'utilisation.
Leggere attentamente le istruzioni per l'uso.

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Ingredients: Aqua, Undecylenamidopropyl Betaine, Polyaminopropyl Biguanide (Polyhexanide)

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GB Instructions for use Prontoderm® C

Aqueous ready to use solution for the antimicrobial cleaning of skin areas e.g. during urinary catheterisation. Reduces odour.

Ingredients: aqua, undecylenamidopropyl betaine, polyaminopropyl biguanide (polyhexanide)

Application: UROLOGY: Moisten two sterile swabs with Prontoderm® C. Vagina: Wipe the genital area with the first swab, taking care to wash the vagina from front to back. Penis: Pull the foreskin back and wash the head of the penis carefully with the swab. With the second swab wipe around the urethral opening and leave for at least 1 minute. Afterwards the catheter can be inserted. It should not be rinsed with water. Cleansing of the entry port for a suprapubic catheter should be carried out in the same manner as the cleansing of an entry port for PEG tubes.

PEG TUBES: For cleansing the entry port after the incision site has healed (at the earliest from the seventh day post-op). Apply Prontoderm C onto the PEG-Site. If there is any encrustation, allow at least one minute for the Prontoderm® C to take effect. Afterwards dry the skin area around the entry site well by dabbing with sterile gauze and covering with a sterile dressing.

STOMA: Soak some swabs with Prontoderm® C and clean around peristomal skin carefully. Dry well with a dry swab then apply pouch as required.

Special precautions: Residues from soap and cleansing lotions should be removed with water before the use of Prontoderm® C. The addition of other products at the same time as Prontoderm® C must not be allowed.

Note: Intended for external use only. Not for oral use! Non-returnable bottles! Do not transfer contents or refill.

Use by date: Opened bottles should be used within two months after opening.

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