






THE NEW REFERENCE IN SUPRAPUBIC CATHETERIZATION



Ready to Use system:

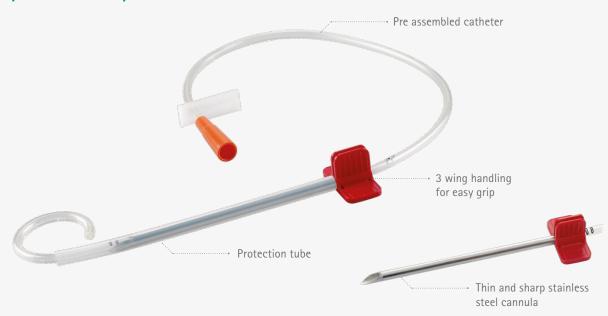

Ready to Use system:

The catheter is already inserted into the cannula

Saves time

The protection tube can remain in place until use.

• Increases safety and reduces the risk of sharp injury.

Supra Pubic Catheterization participates to the prevention of Nosocomial UTI

"There is a lower rate of infection in those with a suprapubic rather than urethral catheters despite the former being used for two weeks or longer"

EAUN guideline 2012. Evidence-based Guidelines for Best Practice in urological health care. Catheterisation. Indwelling catheters in adults. Urethral and suprapubic.

Geng V, Cobussen-Boekhorst H, Farrell J, Gea-Sánchez M, Pearce I, Schwennesen T, Vahr S, Vandewinkel C.

Ready to Use system:



The catheter is already in the cannula. Keep the protection tube on the cannula



Pull the catheter until the tip is right below the cannula bevel.



Remove the protection tube to begin the puncture.

The standard in suprapubic catheterization



New silicone integral balloon catheters:




Low profile balloons:

- Designed for easier removal with reduced risk of cuffing.¹
- Designed for easier replacement without ridge for better insertion.

Silicone material:

Can be used and replaced over long periods of time²

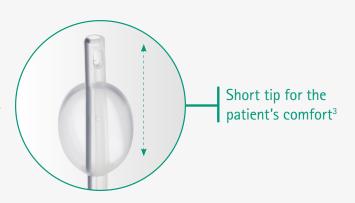

(1) Br. J. of Cummunity Nurs, 2005 Oct; 10 (10): 461-464, Suprapubic Catheterization: Challenges in changing catheters. Robinson J. (2) 5th International Consultation on Incontinence, Paris 2012. Committee 20: 1519-1642, Management using continence products

Extended range of integral balloon catheters

from CH10 to CH20, for punctures and replacements



Extended range of dilation sets: up to CH16

with color code



New convenient packaging

Boxes of 10 units and 5 units for an easier management of stock in the hospital

Boxes of 1 unit for replacement in the community.



Punction sets with balloon for all use. New range Cystofix® Ready-to-Use

Catheter design	Catheter material	Cannula lengh	Cannula diameter	Catheter size	Catheter length	Box of	Reference	Composition of sets
Balloon straight tip integral balloon	Silicone	12cm	5,4mm	☐ CH12	44cm	10	4450200	
Ready to Use		12cm	5,4mm	■ CH14	44cm	10	4450220	
Balloon & J tip classical balloon Ready to Use	Silicone	12cm	5,4mm	☐ CH12	53cm	10	4450210	

Punction sets with pigtail for short term use

Catheter design	Catheter material	Cannula lengh	Cannula diameter	Catheter size	Catheter length	Box of	Reference	Composition of sets
Complete set with 2L urine bag Ready to Use	Polyurethane	8cm	3,4mm	■ CH10	65cm	10	4450150	
		12cm	3,4mm	■ CH10	65cm	10	4450160	
		12cm	5,4mm	■ CH15	65cm	10	4450170	□ 📜
Eco set (no bag) Ready to Use	Polyurethane	8cm	3,4mm	■ CH10	65cm	10	4450100	
		12cm	3,4mm	■ CH10	65cm	10	4450120	
		12cm	5,4mm	■ CH15	65cm	10	4450130	Ŷ
TUR set	Polyurethane	12cm	5,4mm	■ CH15	65cm	10	4441354	
Paediatric complete set with 2L urine bag. Ready to Use	Polyurethane	5cm	3,4mm	■ CH10	50cm	10	4450180	
Minipaed punction	Polyurethane	5cm	2,2mm	☐ CH05	50cm	10	4441036	

Exchange sets with integral balloon catheter

Set design	Catheter design	Catheter material	Catheter size	Catheter length	Box of	Reference	Composition of sets
Dilation and replacement. With guidewire and dilator	Balloon straight tip	Silicone	□ CH12	44cm	5	4450512	
			■ CH14			4450514	
			CH16			4450516	
Replacement with guidewire	Balloon straight tip	Silicone	■ CH10	- 44cm	5	4450010	
			☐ CH12			4450012	
			■ CH14			4450014	
			CH16			4450016	
	Balloon straight tip	Silicone	☐ CH12	44cm	5	4450712	
5			■ CH14			4450714	
Replacement without guidewire			CH16			4450716	
			■ CH18			4450718	
			CH20			4450720	
Replacement single pack	Balloon straight tip	Silicone	☐ CH12	44cm	1	4450812	
			■ CH14			4450814	
			CH16			4450816	
			■ CH18			4450818	
			CH20			4450820	

Manufactured and distributed by:

B. BRAUN MEDICAL | 26 Rue Armengaud | 92210 Saint-Cloud - France

Tél. 01 41 10 53 00 | Fax 01 41 10 53 99

B. Braun Medical | SAS with a share capital of 31 000 000 € | RCS Nanterre 562050856 www.bbraun.fr

www.bbraun.com/continence-care-and-urology

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.

Non binding documents and photographs.