

Medibox® – How to use

LID ASSEMBLY







Place the lid on the medical waste container and press firmly on the opposite corners of the lid until a "click" confirms the lock.

Check that the lid is properly locked on all sides!


DISPOSAL



Open the lid by applying slight pressure on the tab – the lid flap jumps up slightly.



Push up the lid flap with one hand, until it locks into a standing position.

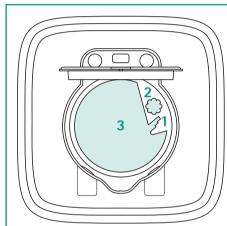

DISPOSAL OF

- 1. luer and luer-lock needles
- 2. pen needles
- 3. various medical sharps



Disconnect luer and luer lock connections between syringe and cannula.



Unscrew a pen needle from a pen.




Dispose large medical sharps.

LID CLOSURE



TEMPORARY CLOSURE: Release the lid flap from the standing position and press it lightly onto the housing.



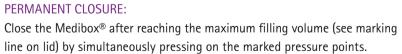



Dispose the permanently closed Medibox® in accordance with your local regulations.

In order to view the Medibox® handling video directly on your mobile phone and to obtain product information, simply scan the QR code with the camera of your smartphone and a suitable reader application and get to the website.

