



Meliseptol® HBV Tissues

PRACTICAL AND QUICK

Meliseptol® HBV Tissues

PRACTICAL AND QUICK

PROPERTIES

- Practical and quick with high quality wipes
- For wipe disinfection of small alcohol-resistant surfaces
- Impregnated with Meliseptol® rapid liquid (For features of Meliseptol® rapid, see page 10)
- Dispenser box for practical, safe and hygenic removal
- Reusable box with refilling pack
- Dermatologically tested
- Save-lock with audible click
- Top with guiding rail & strong back flap
- 90 days of in-use stability

IN A NUTSHELL

- Fast and effective within 1 min.*
- Aldehyde-free
- Box with refill pack
- Listing: the DGHM/VAH list and the IHO-virucide list

APPLICATION INSTRUCTIONS

Perform hand disinfection and wear gloves before use.

Follow Occupational Safety and Health Standards for handling of flammable and combustible liquids. Electrical appliances must be switched off before being disinfected. If this is not possible, precautions must be taken to ensure that no switching occurs while disinfection is taking place. Let hot surface areas cool down before disinfection. Remove all visible contaminants prior to the disinfection. Wipe the visible clean surface with Meliseptol® HBV Tissues so that it is completely wet. Keep the surface wet until theexposure time has elapsed. Do not wipe the surface dry. For the correct exposure times see enclosed table.

DISINFECTION WIPES - QUICK AND ECONOMIC

Refill the empty, clean, disinfected and dry Meliseptol® HBV Tissues box with the refilling pack with flexible foil.

Physico-chemical data ready-to-use solution

 Appearance
 clear, colorless liquid

 pH-value
 approx. 7

 Density (20 °C)
 ca. 0.98 g /ml

 Flash point (DIN 51 755)
 + 31 °C

Meliseptol® HBV-Tissues - composition:

One dispenser or refill pack contains 100 ready-to-use wipes, impregnated with 180 g Meliseptol* rapid solution. Ingredients in accordance with the Regulations on Detergents EC 648/2004: < 5 % non-ionic surfactants. Labelling of dangerous goods: see material safety data sheet (MSDS) Cautions: Use disinfectants safely. Always read the label and the product information before use.

Spectrum of activity	Test Norms	Exposure Time
Surface disinfection Bacteria, yeasts (C. albi- cans) and enveloped viruses (incl. HBV, HCV, HIV, Vacci- nia virus, BVDV)	DGHM/VAH 09/2001 (corr. EN 13727, EN 13624, EN 13697) EN 1040, EN 1275, EN 1276, D V V/RKI*	1 min.
4-field test (Bacteria & yeasts)	EN 16615	1 min.
Mycobacteria (TbB)	DGHM 09/2001 (corr. EN 14348)	30 sec.
Fungi (A. brasiliensis)	EN 13624	15 min.
Non-enveloped viruses	D V V / RKI* carrier test	10 min.
Avian Influenza A	D V V/RKI*	15 sec.
HBV (DHBV)	D V V/RKI*	15 sec.
Rotavirus	D V V/RKI*	1 min.
Adenovirus	D V V/RKI*	1 min.
Polyomavirus	D V V/RKI*	5 min.
Poliovirus	Surface carrier test	10 min.
Norovirus (FCV)	D V V/RKI*	2 min.
Norovirus (MNV)	EN 14476	1 min.

Expert reports are available upon request

acc. recommendation of RKI, Bundesgesundheitsbl. 01-2004, DW/ RKI suspension test



PRODUCT SIZE	REF
Dispenser box with 100 wipes	18706, 19096
Refilling pack with 100 wipes	18707, 19097

B. Braun Medical AG | Infection Control | Seesatz 17 | CH-6204 Sempach

Phone +41 58 258 50 00 | Fax +41 58 258 60 00 | info.bbmch@bbraun.com | bbraun.com/infection-prevention