



1 CONNECTION OF IV-SET

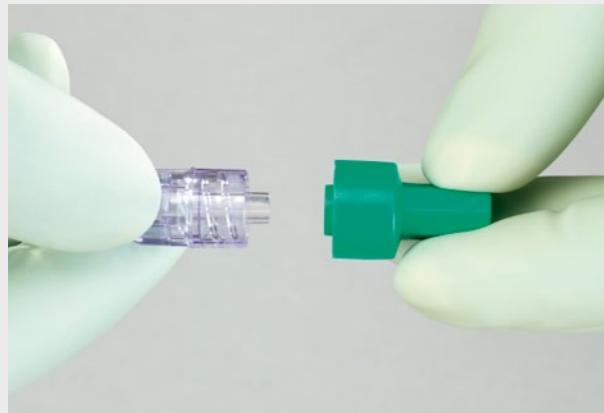

Remove PrimeStop cap.

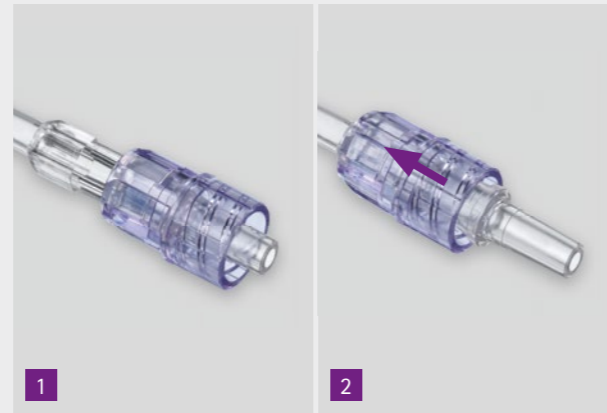

Move collar backwards to activate its rotating function (2). The rotating collar helps to tighten the connection once inserted.



Connect Spin-Lock® connector to counterpart, such as Caresite® extension line.



Move collar forward and screw it clockwise onto the counterpart.

2 DISCONNECTION OF IV-SET



Unscrew the collar counter-clockwise.



Move collar backwards up to the stop position.



Turn the collar counter-clockwise to disconnect. The hexagonal design of the stop helps to make the disconnection easier.

Spin-Lock® Connector

Rotating collar for easy and tight connection